

Configuration

Configuration Manager

The credentials needed to make a connection to your Pathomation account are stored in Adobe Experience Manager's "Configuration Manager".

Go to <http://localhost:4502/system/console/configMgr> (Or any other localhost:xxxx you are currently running with to host your AEM instance).

Look for "Pathomation Configuration"


Here you can set the url, username and password.

Configuration for a demonstration account is:

url <https://host.pathomation.com/pma.core.2/>

username PMA_UI_demo

password PMA_UI_demo

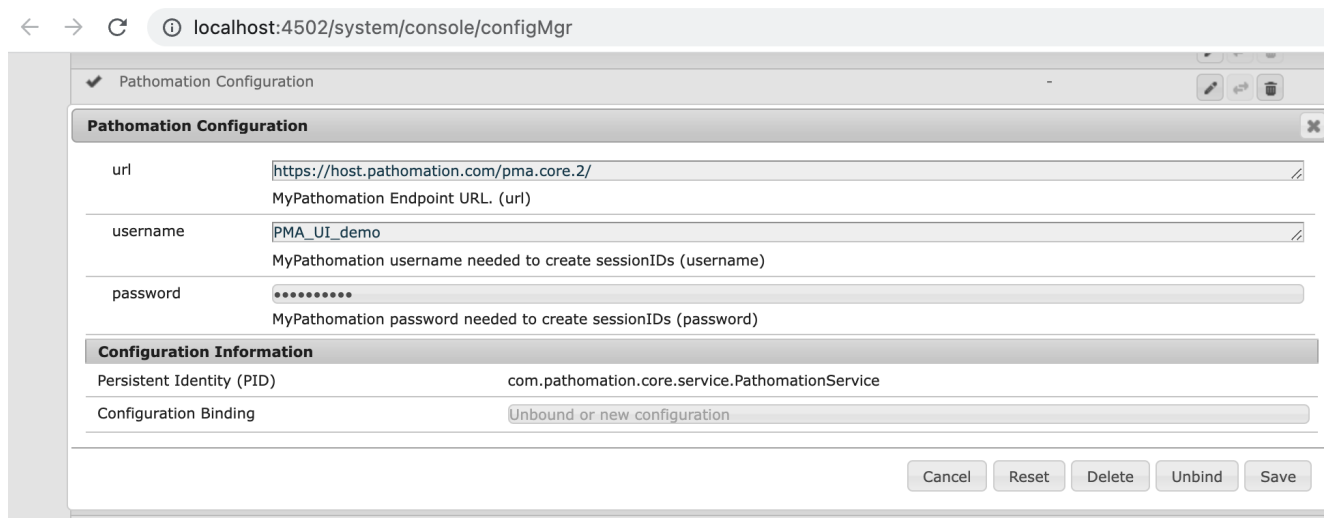

Editing your Template


In order to have the "Pathomation Virtual Slide Component" as an available component on your site editor to use, you need to include it in the template that your current page is using

Go to the template editor and edit the specific template that you'd like to have the component in.

This is a link to the template console where you can edit the Templates: <http://localhost:4502/libs/wcm/core/content/sites/templates.html/conf> (Or any other localhost:xxxx you are currently running with to host your AEM instance).


However, there is a direct link to the template that you use on your current page in the top left corner of the AEM Site Editor (see image below). Click on the "sliders icon" on the top left of your page to open this menu.

"Edit Template" will bring you to the template where you need to add "Pathomation Plugin" to your list of usable components.

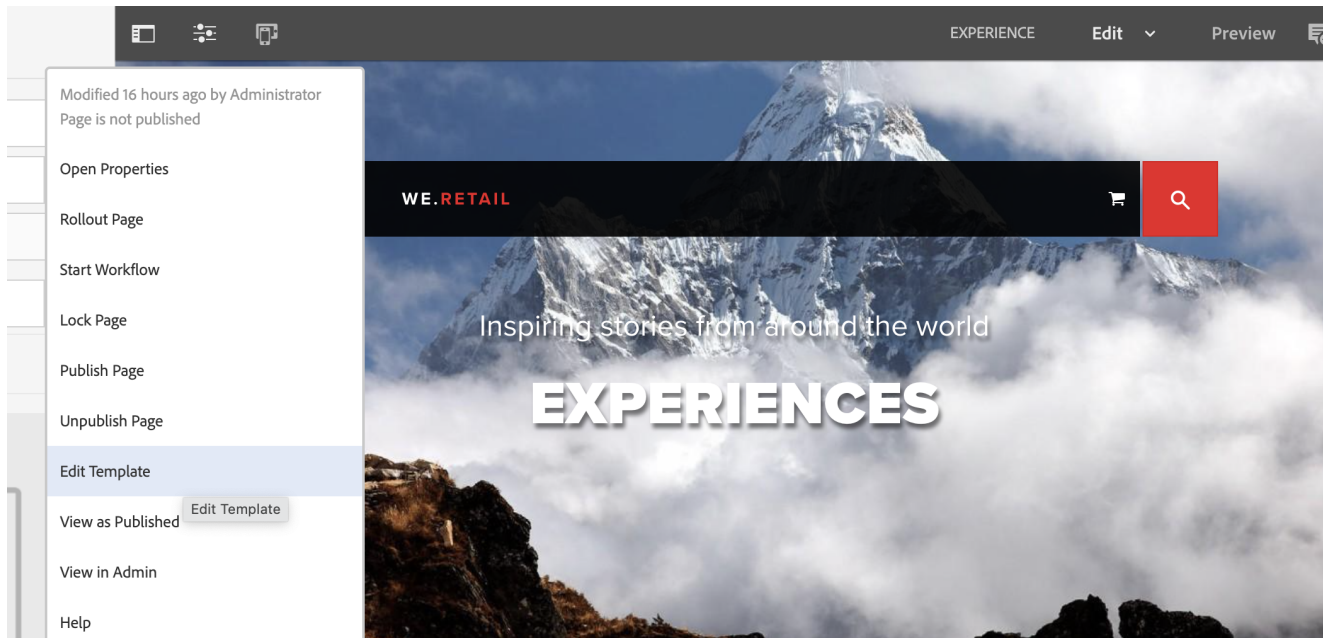

When inside the Template Editor, you see a "Layout Container" in the middle of the template which is used to place the components.

Go to the Policy of this Layout Container by clicking the Policy button on the actionbar (see image below).

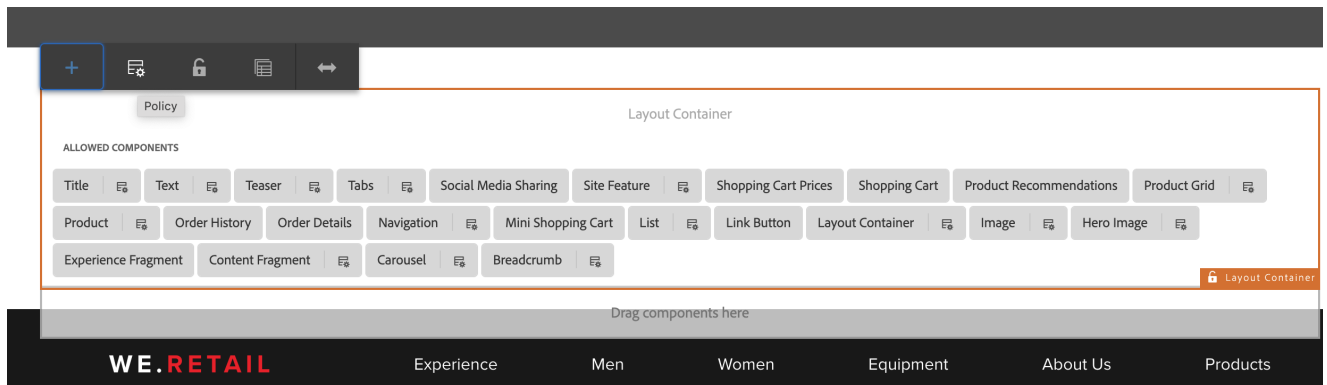

On the menu you now enter, there is a "Properties" menu on the right half of the page.

The tab "Allowed Components" has "Pathomation Plugin" as an option to select.

Either scroll down to find it, or type Pathomation in the search bar.

Mark the checkbox to add this component to your template.

Policy

First choose a policy to apply. Policies can be shared across templates

Select policy

We.Retail Default ▾ + 

Policy Title *

We.Retail Default

Policy Description 

Policy for editable layout containers

Properties

Then adjust the settings of the selected policy to configure the component

Allowed Components Default Components Responsive Settings Styles

×

>

Pathomation Plugin

1

You are done now with editing your template, to see the result: go back to your site editor and refresh the page (If it was open before the template editing of the previous step)

On the components bar you now have "Pathomation Virtual Slide Component" as an option.