

Installation Guide

#ToDo

Write the url where you can download the package

Uploading the package


Go to the Adobe Experience Manager's "Package Manager" to upload and install the Pathomation-plugin-x.x.zip (x holds the version number) file using this link: <http://localhost:4502/crx/packmgr/index.jsp>

(Or any other localhost:xxxx you are currently running with to host your AEM instance).

For compatibility: you need AEM 6.5 with servicepack 6.5.10.

In the top bar, next to the search field you'll find the "Upload Package" button.

Select the "Pathomation Plugin" and upload it.

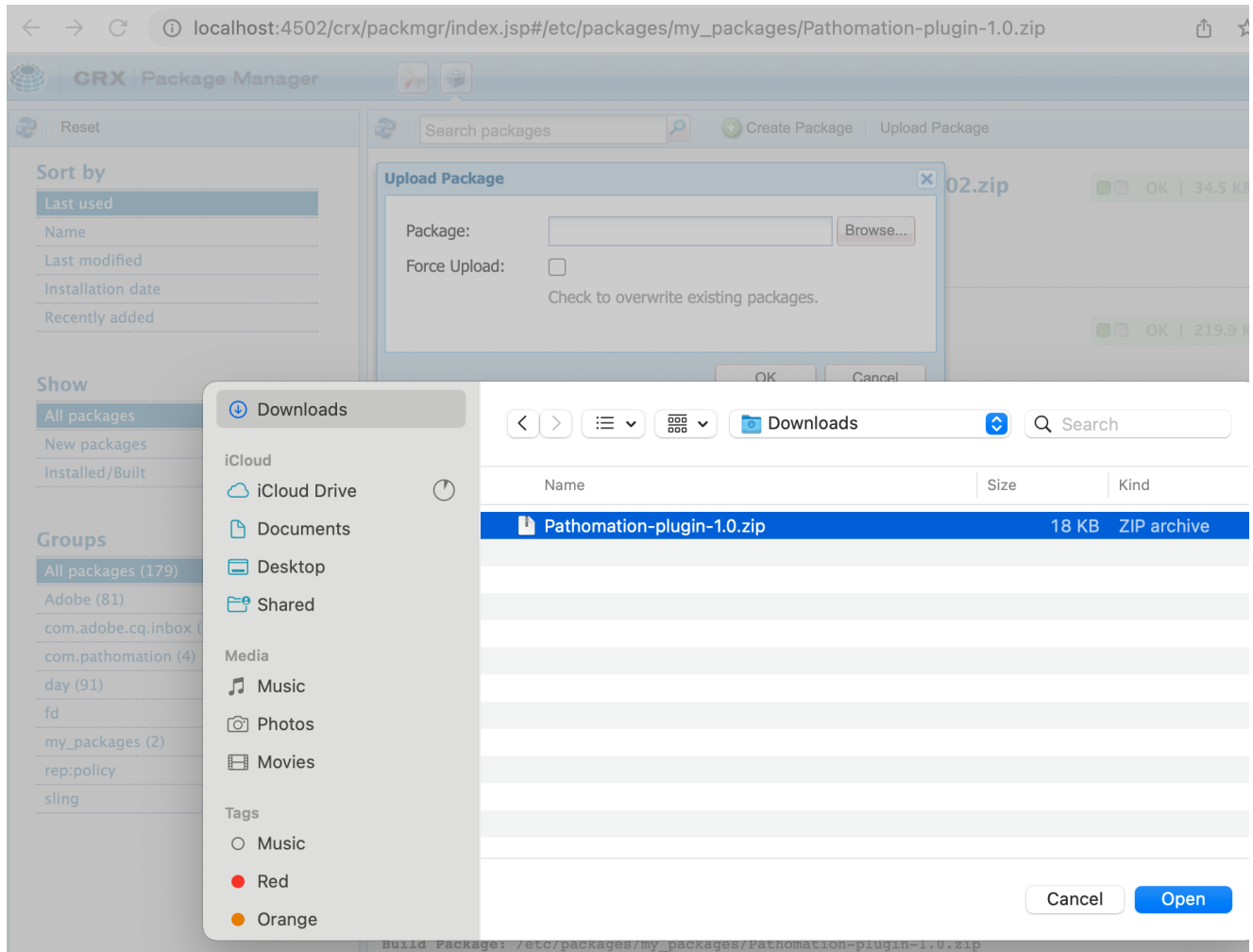

Installing the package

After uploading the package, hit the "Install" button as highlighted in the image below.

CRX Package Manager Reset Search packages Create Package Upload Package Administrator@crx.defau

Sort by

- Last used
- Name
- Last modified
- Installation date
- Recently added

Show

- All packages

Pathomation-plugin-1.0.zip OK | 17.6 KB

Version: 1.0 | Build: 1 | Last built 14:39 | admin

Edit Build Install Download More

Package: Pathomation-plugin

Download: Pathomation-plugin-1.0.zip (17.6 KB)

Group: my_packages

Filters: