


# Moodle Plugin installation guide

1) Download all 4 zip files


1/19/2021	2:04 PM	2364414	<a href="#">PMAmoodle_filter.zip</a>
1/19/2021	2:04 PM	82318	<a href="#">PMAmoodle_pmacase.zip</a>
1/19/2021	2:04 PM	40627	<a href="#">PMAmoodle_pmacasecollection.zip</a>
1/19/2021	2:04 PM	71479	<a href="#">PMAmoodle_pmaslide.zip</a>

<https://host.pathomation.com/plugins/moodle/>

2) Go to the “Install plugins” page

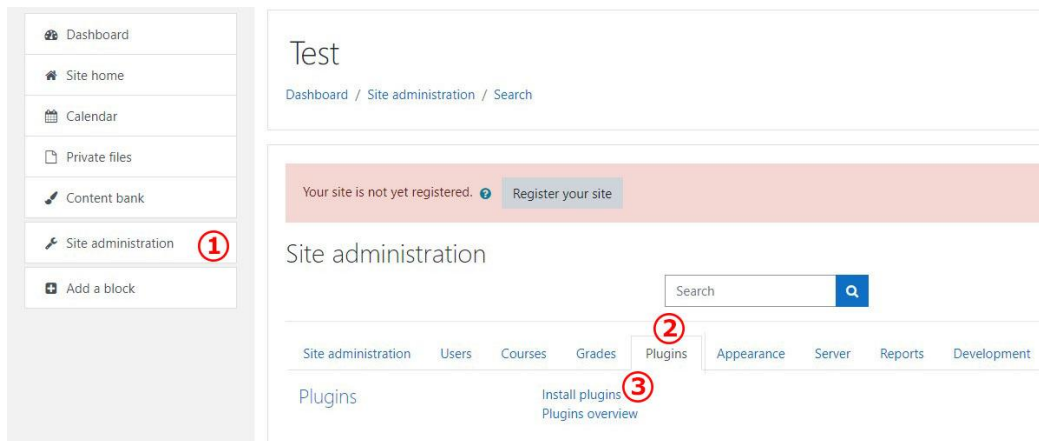

3) Drag & drop the filter zip file

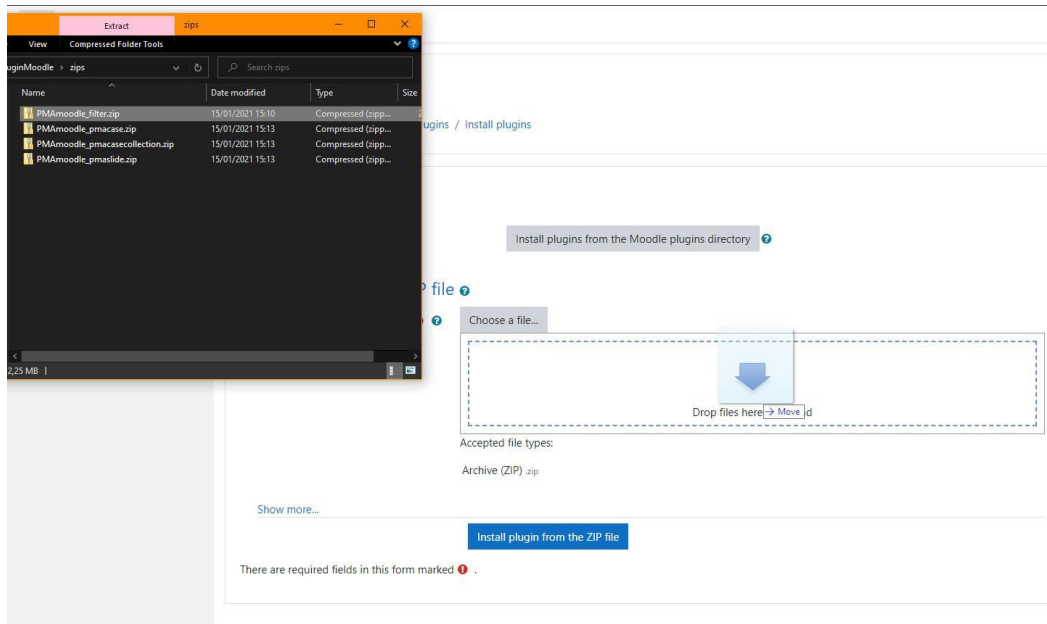

4) Select “Text filter (filter)” option at “Plugin type” dropdown menu and click “Install plugin from the ZIP file”

## Plugin installer


Install plugins from the Moodle plugins directory ?

▼ Install plugin from ZIP file ?

ZIP package ! ?

PMAmoodle\_filter.zip

Accepted file types:  
Archive (ZIP) .zip

Plugin type ?  

✓ Plugin type location C:\inetpub\wwwroot\moodle\filter is writable

Rename the root directory ?

Show less...

There are required fields in this form marked !.

5) If everything is ok, click "Continue"

### Install plugin from ZIP file

Debugging output enabled  
 ZIP C:\inetpub\wwwroot\moodledata\temp\tool\_installdon\60055c64abc6b\plugin.zip  
 Validating filter\_pmaviewer ... OK

[OK] Name of the plugin to be installed [pmaviewer] ?

[OK] Plugin version [2017112804]  
 [OK] Required Moodle version [2016051900]  
 [OK] Full component name [filter\_pmaviewer]

[Warning] Declared maturity level [MATURITY\_BETA] ?

[OK] Plugin release [v1.0]  
 [Debug] Found language file [filter\_pmaviewer]  
 [OK] Write access check [C:\inetpub\wwwroot\moodle\filter]

Validation successful, installation can continue

php_extension	json	<span>!</span> must be installed and enabled	<input type="button" value="OK"/>
php_extension	hash	<span>!</span> must be installed and enabled	<input type="button" value="OK"/>
php_extension	fileinfo	<span>!</span> must be installed and enabled	<input type="button" value="OK"/>
php_setting	memory_limit	<span>!</span> recommended setting detected	<input type="button" value="OK"/>
php_setting	file_uploads	<span>!</span> recommended setting detected	<input type="button" value="OK"/>
php_setting	opcache.enable	<span>!</span> recommended setting detected	<input type="button" value="OK"/>

### Other checks

Information	Report	Plugin	Status
site not https	<span>!</span> if this test fails, it indicates a potential problem It has been detected that your site is not secured using HTTPS. It is strongly recommended to migrate your site to HTTPS for increased security and improved integration with other systems.		<input type="button" value="Check"/>
Your server environment meets all minimum requirements.			<input type="button" value="X"/>

## Plugins check

This page displays plugins that may require your attention during the upgrade, such as new plugins to be installed, plugins to be upgraded, missing plugins etc. Additional plugins are displayed if there is an available update for them. It is recommended that you check whether there are more recent versions of plugins available and update their source code before continuing with this Moodle upgrade.

Check for available updates

### Plugins requiring attention

Cancel new installations (1) [Plugins requiring attention](#) 1 [All plugins](#) 415

Plugin name / Directory	Current version	New version	Requires	Source / Status
<b>Text filters</b>				
PMA_Core slide viewer <small>/filter/pmaviewer</small>		2017112804	• Moodle 2016051900	<b>Additional</b> <b>To be installed</b> Cancel this installation

 Reload

[Upgrade Moodle database now](#)

## Upgrading to new version

filter\_pmaviewer

Success

[Continue](#)

## 6) Configure the plugin and click “Save changes”

### Test

The settings shown below were added during your last Moodle upgrade. Make any changes necessary to the defaults and then click the 'Save changes' button at the bottom of this page.

#### New settings - PMA.Core slide viewer

**Start tag**  
filter\_pmaviewer | starttag  Default: {{PMASLIDE:}}

Tag that marks the start of the content

**End tag**  
filter\_pmaviewer | endtag  Default: }}

Tag that marks the end of the content

**Connection type**  
filter\_pmaviewer | connectiontype  Default: My Pathomation

Preferred connection type

**My Pathomation username**  
filter\_pmaviewer | mypathomationusername  Default: admin

The username of your My Pathomation account

This setting may be hidden, based on the value of **filter\_pmaviewer | connectiontype**.

**My Pathomation password**  
filter\_pmaviewer | mypathomationpassword

The password of your My Pathomation account

This setting may be hidden, based on the value of **filter\_pmaviewer | connectiontype**.

**The PMA.core host url**  
filter\_pmaviewer | pmacorehost  Default: https://host.pathomation.com/pma.core.2/

The url of a PMA.core server to fetch slides from

This setting may be hidden, based on the value of **filter\_pmaviewer | connectiontype**.

**The PMA.control host url**  
filter\_pmaviewer | pmacontrolhost  Default: https://host.pathomation.com/pma.training/

The url of a PMA.control server to fetch case collections from

This setting may be hidden, based on the value of **filter\_pmaviewer | connectiontype**.

**PMA.core username**  
filter\_pmaviewer | pmacoreusername  Default: admin

The username to authenticate to the PMA.core server

This setting may be hidden, based on the value of **filter\_pmaviewer | connectiontype**.

**PMA.core password**  
filter\_pmaviewer | pmacorepassword

The password to authenticate to the PMA.core server

This setting may be hidden, based on the value of **filter\_pmaviewer | connectiontype**.

[Save changes](#)

- Install the rest zip files (pmaslide, pmacase and pmacasecollection) by setting “Plugin type” as “Atto HTML editor / Atto plugin (atto)” and following previous steps

Plugin installer

Install plugins from the Moodle plugins directory ?


▼ Install plugin from ZIP file ?

ZIP package Choose a file...

PMAmoodle\_pmaslide.zip

Accepted file types:

Archive (ZIP) .zip

Plugin type Atto HTML editor / Atto plugin (atto) ⌵ 

✓ Plugin type location C:\inetpub\wwwroot\moodle\lib\editor\atto\plugins is writable


Rename the root directory

Show less...

Install plugin from the ZIP file

There are required fields in this form marked !.


- Go to the “Manage filters” page and set “PMA.Core slide viewer” active state to “On”

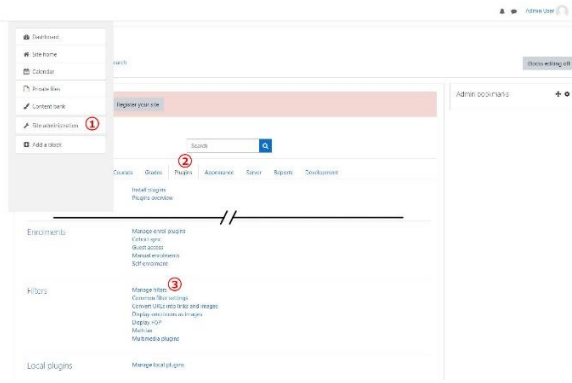

The screenshot shows the Moodle administration interface. The left sidebar contains navigation options like Dashboard, Site home, Calendar, Profile, Course bank, Site administration, and Add a block. The main content area is titled 'Manage filters' and shows a list of filters. The 'PMA.Core slide viewer' filter is highlighted with a red arrow. The filter's 'Active' state is currently set to 'Off'.

Active?	Order	Apply to	Settings	Uninstall
On	1	Content	Settings	Uninstall
On	2	Content	Settings	Uninstall
On	3	Content	Settings	Uninstall
On	4	Content	Settings	Uninstall
On	5	Content	Settings	Uninstall
On	6	Content	Settings	Uninstall
Disabled	7	Content	Settings	Uninstall
Disabled	8	Content	Settings	Uninstall
Disabled	9	Content	Settings	Uninstall
Disabled	10	Content	Settings	Uninstall
Disabled	11	Content	Settings	Uninstall
Disabled	12	Content	Settings	Uninstall
Disabled	13	Content	Settings	Uninstall
Disabled	14	Content	Settings	Uninstall
Disabled	15	Content	Settings	Uninstall
Disabled	16	Content	Settings	Uninstall
Disabled	17	Content	Settings	Uninstall
Disabled	18	Content	Settings	Uninstall
Disabled	19	Content	Settings	Uninstall
Disabled	20	Content	Settings	Uninstall

- Go to the “Manage editors” page and enable “Atto HTML editor”, if not enabled by default

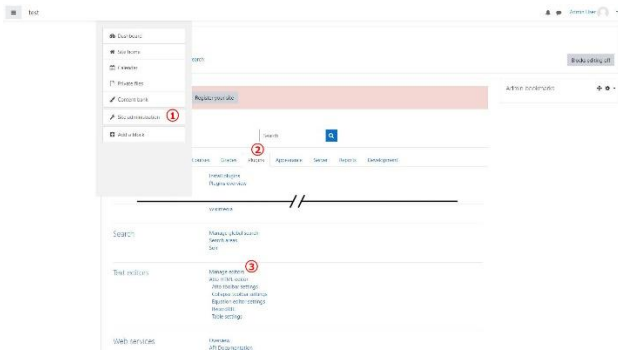


### Manage editors

Available text editors

Name	Enable	Up/Down	Settings	Uninstall
Atto HTML editor	<input checked="" type="checkbox"/>		Settings	Uninstall
TinyMCE HTML editor	<input type="checkbox"/>		Settings	Uninstall
Plain text area	<input type="checkbox"/>			

Please choose the editor plugins you wish to use and arrange them in recommended order.  
Changes in table above are saved automatically.

10) Go to the “Atto toolbar settings” page and in “Toolbar config” textbox add line “slides = pmaslide, pmacase, pmacasecollection”, saving changes in the end

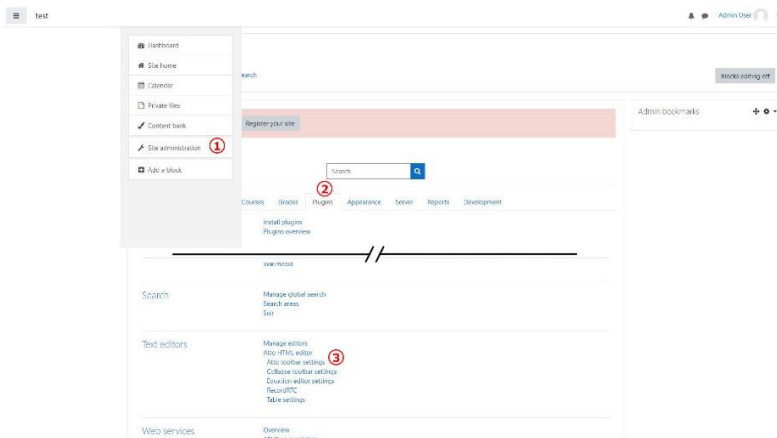

Changes in table above are saved automatically.

Toolbar config  
editor\_atto | toolbar

```
collapse = collapse
style1 = title, bold, italic
list = unorderedlist, orderedlist, indent
links = link
slides = pmaslide, pmacase, pmacasecollection ←
files = emojiPicker, image, media, recordrtc, managefiles, h5p
style2 = underline, strike, subscript, superscript
align = align
```

Default:  
collapse = collapse  
style1 = title, bold, italic  
list = unorderedlist, orderedlist, indent  
links = link  
files = emojiPicker, image, media, recordrtc, managefiles, h5p  
style2 = underline, strike, subscript, superscript  
align = align  
insert = equation, charmap, table, clear  
undo = undo  
accessibility = accessibilitychecker, accessibilityhelper  
other = html

The list of plugins and the order they are displayed can be configured here. The configuration consists of groups (one per line) followed by the ordered list of plugins for that group. The group is separated from the plugins with an equals sign and the plugins are separated with commas. The group names must be unique and should indicate what the buttons have in common. Button and group names should not be repeated and may only contain alphanumeric characters.

Autosave frequency  
editor\_atto | autosavefrequency

1 minutes Default: 1 minutes

This is the number of seconds between auto save attempts. Atto will automatically save the text in the editor according to this setting, so that text can be automatically restored when the same user returns to the same form.

**Save changes**